



1. Remove the Myriad Clinical Pack Overwrap bag and mailing label.

2. Place signed **TEST REQUEST FORM A**, **INFORMED CONSENT FORM B** (required for Medicare patients), copies of insurance cards, and labeled collection tube in the **Oragene[®]** box into the test box.



3. Close the test box and place it into the Myriad Clinical Pack Overwrap bag, and affix the enclosed mailing label onto the front of the bag.*



4. If you don't have a regularly scheduled FedEx pickup, call 800-GO-FEDEX (800-463-3339) to request one. Press "0" and specify that you are shipping with a prepaid air bill.

Please do not write the patient's name on the test box.

Note: Immediately following specimen collection ship the sample via FedEx. Samples can be held at ambient room temperature for up to 7 days prior to receipt at Myriad.

Once the test kit is received, Myriad will automatically replenish your kit supply. If you would like to order additional tests or have any questions about your sample submission, please call the Myriad Customer Service Department at 800-4-MYRIAD (800-469-7423).

*It is the shipper's responsibility to ensure that the package containing a diagnostic specimen conforms to FedEx guidelines along with all applicable local, state, federal and international regulations.